



Kindred

Kindred Series Case Goods

All of the surfaces listed below will withstand cleaning using a solution of 0.5 Hydrogen Peroxide (such as Virox®) or a 70% Isopropyl alcohol solution following the recommended cleaning frequency of a patient room environment as set out in such guidelines as Best Practices for Environmental Cleaning by PIDAC. When cleaning with either of the above solutions, it is a requirement that the solution not be allowed to sit for an extended time on the surface, but should be wiped thoroughly within a few moments of application.

Laminate Surfaces

Dust with a soft, clean cloth. To remove fingerprints etc. scrub with a damp soapy cloth followed by a rinse with cold water. Stubborn stains can be removed with hot water and a bit of vinegar. Avoid using abrasive cleaning tools as these may scratch the laminate surface. It may be necessary to use a mild non-abrasive cleaner.

Thermofoil Surfaces

Avoid using harsh cleaners with abrasive additives. Also avoid bleach and any solvent-based cleaners, as well as any cleaner containing ammonia.

Metal Drawer Components

Dust with a soft, clean cloth. It may be necessary to use a mild non-abrasive cleaner. Do not use abrasive Ammonia-based cleaners. Mild non-abrasive cleaners should only be used occasionally.



Casters (if applicable)

Casters should be regularly inspected and cleaned of debris that will impede their ability to roll and swivel.

Drawer Liners (if applicable)

Drawer liners can be removed from drawer when drawer is fully extended. Drawer liners are a seamless plastic construction and can be cleaned/disinfected using strong cleaners and abrasive cleaning tools if necessary.